



DESCRIPTION:

THE DRAIN ADAPTER IS DESIGNED TO BE USED WITH THE UNIVERSAL RECESSED VALVE HOUSING (DIALYSIS SUPPLY BOX). THE ADAPTER PROVIDES A DRAIN FOR THE DIALYSIS MACHINE, AS WELL AS A MECHANISM TO COLLECT AND DIVERT TO DRAIN ANY DRIPPINGS OR SPILLAGE COLLECTED IN THE BOTTOM OF THE DIALYSIS SUPPLY BOX.

INSTALLATION INSTRUCTIONS:

DRILL A 1 1/16" HOLE THROUGH THE BOTTOM OF THE SUPPLY BOX TO ACCOMODATE THE 3/4" INCH NIPPLE DRAIN ADAPTER. APPLY TEFLON TAPE TO THE THREADS ON THE TOP PORTION OF THE ADAPTER. APPLY A BEAD OF SILICONE TO THE TOP EDGE OF THE COLLECTION PORTION OF THE ADAPTER. FROM THE UNDERSIDE OF THE SUPPLY BOX (INSIDE THE CHASE OR CASEWORK), INSERT THE DRAIN ADAPTER ELBOW ONTO THE ADAPTER. FIRMLY TIGHTEN THE ELBOW, SECURING THE DRAIN ADAPTER TO THE SUPPLY BOX. DIRECTLY BEHIND THE ELBOW, DRILL A 3/16" HOLE. THIS HOLE WILL SEND ANY SPILLAGE OR DRIPPING TO THE DRAIN; THEREFORE, BE CERTAIN IT IS DRILLED ABOVE THE COLLECTION PORTION OF THE ADAPTER. A HOSE BARB ADAPTER OR ANY TYPE OF CONNECTOR (NOT SUPPLIED) CAN BE SCREWED INTO THE ELBOW FOR THE DIALYSIS MACHINE DRAIN LINE.

NOTE:  
UNLABELED ARROWS  
REPRESENT WASTE  
WATER FLOW.

DRAWING NAME: DRAIN ADAPTER DETAIL	
PART NUMBER: DEP-02	
REVISION: A	REVISION DATE: 01-22-11

**DIALYSIS**

◆

ENHANCEMENT PRODUCTS

Phone: 1-760-789-7404 | Fax: 1-760-789-1521  
www.dialyisep.com